## ULTRASTOP pro med. Lösung Monodose

### ANTIBESCHLAGMITTEL

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#### AREAS OF USE

**ULTRASTOP pro med.** Solution Monodose is especially suitable for:

- ♦ use in **strictly aseptic endoscopy**, which has to be performed with sterile apparatuses in visceral cavities (e.g. laparoscopy, thoracoscopy, arthroscopy etc.) as well as for
- ♦ all medical examination mirrors and lenses (e.g. larynx and epipharynx mirrors, stroboscopes, dental mirrors, colonoscopes, etc.).

Medical examination mirrors, endoscopes and similar devices used in aseptic surgery as well as microscope oculars and lenses can be easily, simply and reliably protected from fogging. Pre-warming the optics is no longer necessary, which is time- and labour-saving for the doctor.

**Note:** **ULTRASTOP pro med.** can be used on glass and nearly all synthetic materials. If other coatings have been used or if the coating is of doubtful resistance, please contact the manufacturer or the retailer of the object to be treated.

#### INSTRUCTIONS FOR USE

Strictly aseptic use.

Open the sachet along the perforation; if necessary the bottle can be handled sterile.

- ♦ Application to hand-held instruments (e.g. stroboscopes, hand endoscopes etc.). Open the **ULTRASTOP pro med.** bottle and drop the contents onto a suitable surface, e.g. the Sugi® Endopad included in the application kit. To clean and moisten the lens of the hand-held instrument it should be thoroughly rotated in the moist sponge. Leave it to act for a few seconds and after evaporation, if necessary, gently polish with sterile non-threading gauze. When using Sugi® Endopads, the pad can be applied around the operating area after removing the protective film from the adhesive strip.
- ♦ Application to floor-mounted or fixed appliances, which are permanently fixed or difficult to manoeuvre: Open the **ULTRASTOP pro med.** bottle and drop the contents onto sterile, non-threading gauze. Thoroughly clean and moisten the surface to be treated with the moist gauze. Leave it to act for a few seconds and after evaporation, if necessary, gently polish with sterile non-threading gauze.

**ULTRASTOP pro med.** if used properly, does not smear. However, if too much is used the surface may appear cloudy. In this case the surface should be cleaned with distilled water or sterile gauze, and the above mentioned procedure should be repeated.

 Not for injection! Highly flammable!

Keep out of the reach of children!

**STORAGE CONDITIONS:** Do not store above 25 °C. Protect from light.

#### SHELF-LIFE:

See print on packaging.

 **ULTRASTOP pro med.** Solution Monodose is for single use only.

#### AVAILABLE IN THE FOLLOWING FORMS:

**ULTRASTOP pro med.** Solution Monodose (**sterile**):

1 application kit or 20 application kits consisting of **ULTRASTOP pro med.** Solution Monodose, 1 x 5 ml and 1 Sugi® Endopad, sterile endopad, Ø 34 mm, REF 31782 (CE 0123, manufacturer: Kettenbach GmbH & Co. KG, Im Heerfeld 7, D-35713 Eschenburg)

30 x 5 ml **ULTRASTOP pro med.** Solution Monodose (Clinic)

**ULTRASTOP pro med.** Solution, 30 ml vial (**sterile**)

**ULTRASTOP** Spray, 15 ml

**ULTRASTOP** bottle with dropper pipette, 25 ml

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Stand der Information: April 2019

#### INSTRUCTIONS FOR USE

## ULTRASTOP pro med. Solution Monodose

### ANTIFOG-AGENT

#### INGREDIENTS

**ULTRASTOP pro med.** Solution Monodose contains a special combination of surface-active substances in an alcohol/water matrix (ethanol, purified water, < 5 % surface-active substances). Each Monodose contains 5 ml.

Application kit: **ULTRASTOP pro med.** Solution Monodose, 1 x 5 ml + 1 Sugi® Endopad, sterile endopad, Ø 34 mm, REF 31782.

#### PROPERTIES

**ULTRASTOP pro med.** is a high-quality sterile antifog agent for medical lenses. Examination mirrors, endoscopes and similar devices for aseptic surgical use such as microscope oculars and lenses can also be simply, easily and reliably protected from fogging.

The special property of **ULTRASTOP pro med.** is that it reliably prevents fogging of the surfaces of medical lenses.

Surface-active substances in a special combination form an absolutely clear and transparent film on the treated surfaces. This extremely thin film prevents the formation of droplets when water vapour becomes condensed; thus it prevents fogging of the surface.

**ULTRASTOP pro med.** Solution Monodose in the form of sterile single doses ensures simple and safe application. The application kit contains a sterile Sugi® Endopad which also provides the option of working hygienically in an aseptic environment.

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**Hersteller:** MoNo chem-pharm Produkte GmbH, Leystraße 129, A-1200 Wien
**Vertrieb:** SIGMAPHARM Arzneimittel GmbH, A-1200 Wien