Uro-Tainer®

NaCl 0,9 % B. Braun

CZ	Informace k použití	2
DE	Gebrauchshinweise	4
DK	Brugsanvisning	6
ES	Instrucciones de uso	8
FI	Lue tämä ohje ennen tuotteen käyttöä	10
FR	Mode d'emploi	12
GB	Information for Use	14
HU	Használati utasítás	16
IT	Informazioni per l'uso	18
NL	Gebruiksaanwijzing	20
NO	Bruksinformasjon	22
PT	Informações de Utilização	24
SE	Bruksanvisning	26
SK	Informácie o použití	28

schwarz

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Lätus - - -

Irrigation Solution___190_GIF
190/NU19090/0212
GIF (A5-Booklet)
Standort Crissier

B | BRAUN

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Složení

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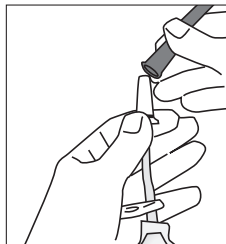
Uro-Tainer® NaCl 0.9 % B. Braun

Maintenance solution for urethral and
suprapubic catheters
Sterile, free of endotoxins

Composition

100 ml solution contains:
Sodium chloride 0.9 g
in Water for Injections

Description

A solution for indwelling and suprapubic urinary catheters. to aid removal of deposits, tissue waste and clots.

Uro-Tainer® is a ready-to-use, disposable system. It consists of a PVC sachet, in a protective plastic overwrapping, containing 50 ml or 100 ml of sterile solution, a flexible tube fitted with a clamp, and a sterile universal connector protected by a plastic cap with a tamperproof seal.

Dosage

As required.

Overdosage

No clinical data exists concerning overdosage.

Important information

- This solution is intended for urinary catheter rinsing only : not for intravenous infusion.
- Instil the solution by gravity feed, avoid force.
- Precautions should be taken when operating on patients with spinal injuries because of the risk of autonomic dysreflexia.
- If the product is not well tolerated appropriate measures should be taken (see secondary effects section).
- To incorporate medication, use Uro-Tainer "M" specially designed to facilitate the addition of drugs.
- Apply using aseptic technique. A port disinfection before use is recommended.
- Only to be used if the contents and the sealing cap are intact, and the solution is clear.
- Store out of the reach of children.
- Refer to the illustrated instructions for use on the next page.

- This is a single-use system. After opening, it must be used and disposed of promptly.
- Discard properly after use.
- Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.
- Animal experiments have shown DEHP to be potentially toxic to reproduction. Given the current state of scientific knowledge, a risk (in case of long-term use) especially for premature male babies cannot be completely excluded. As a precaution, the application of medical products containing DEHP should be restricted to short-term use for pregnant women, breastfeeding mothers, infants, and children.

Secondary effects

This solution is not known to have any secondary effects.

If the liquid exits from the urethra during the irrigation, the catheter may no longer be in the bladder and the patient may require a new catheter.

If the liquid does not flow, the catheter may be kinked in the bladder or it may be blocked and may require changing.

Expiry date

Use before the date marked EXP on the back of the sachet.

Storage

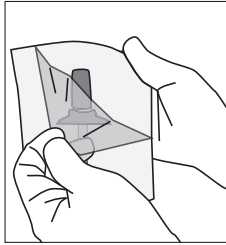
Store in the original packaging, away from light and at a temperature not exceeding 25 °C.

Latest Information update


06-2010

The Uro-Tainer® sachet is supplied sterile and should not be removed from its overwrapping until required for use. If desired, bring solution to body temperature by immersing the wrapped sachet in warm water.


1. Tear open overwrapping and remove Uro-Tainer® sachet.

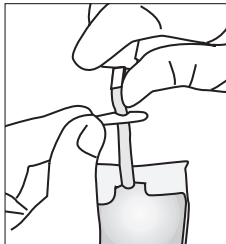
2. Close Uro-Tainer® tube with plastic clamp.

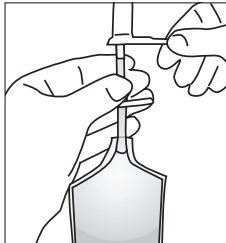
3. Remove tamper proof seal.

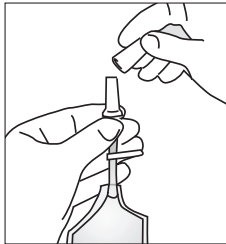4. Twist and withdraw cap without touching sterile connector.

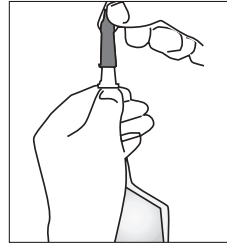5. Insert connector into the catheter funnel.


6. Holding the Uro-Tainer® above the level of the bladder, release the clamp to allow the fluid to drain in by gravity.

7. When the fluid has stopped draining in (this takes a few minutes), lower the Uro-Tainer® below the level of the bladder to drain fluid back into the bag.

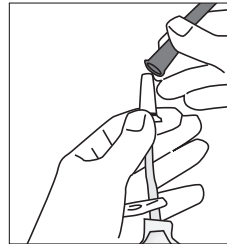8. Clamp tube and remove the connector from the catheter by turning gently. Connect the catheter to a sterile drainage bag. Dispose of Uro-Tainer® sachet and wash hands.

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B. Braun Medical AG