Instructions for use

Breastmilk Bottle with Teat

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EN

Breastmilk Bottle with Teat

Ideal for collecting, storing and feeding breastmilk

Make sure breastfeeding is well established before introducing a teat.

Note: Only use Medela original accessories.

Cleaning and Hygiene

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Take care not to damage parts when using a brush.
- Inspect all items before each use.
- Replace parts if damaged or worn out.
- For hygiene reasons, we recommend replacing teats after 3 months.
- Note: Food colorings may discolor components which will not affect performance.
- Follow these cleaning instructions unless you are told otherwise by your healthcare professional.
- Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant ("sterilizing solution") for longer than recommended, as this may weaken the teat.

Prior to first use and once daily

- Disassemble, wash and rinse all parts.
- Place parts in a saucepan of sufficient capacity.
- Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- Heat it up and allow to boil for 5 minutes.
- Remove from stove and drain water off.
- Dry parts with a clean towel.

As an alternative to boiling: Dishwasher: Clean parts in top rack.

Microwave: Use Medela Quick Clean™ Microwave Bags or other microwave-safe device.

Steam sterilizer: Follow the manufacturer’s instructions.

After each use

- Disassemble and wash all parts in warm, soapy water.
- Rinse in clear water. Dry parts with a clean towel.
- After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid.
- Alternatively, the parts can be wrapped in a clean paper or cloth towel.
- Do not place in direct sunlight.
- Avoid touching the inside of bottles and lids.

Storing and thawing breastmilk

Storage guidelines for freshly expressed breastmilk (For healthy term babies)	
Room temperature	4 – 6 hours at 19 to 26 °C (66 to 78 °F)
Refrigerator	3 – 8 days at 4 °C (39 °F) or lower
Freezer	6 – 12 months –18 to –20 °C (0 to 4 °F)
Thawed breastmilk	In refrigerator for 10 h max. Do not refreeze!

- Do not store breastmilk in the door of the refrigerator. Instead choose the coldest part of the refrigerator (this is at the back of the glass shelf above the vegetable compartment).
- You can freeze expressed breastmilk in milk bottles or "Pump & Save™" bags. Do not fill the bottles or bags more than 3/4 full to allow space for possible expansion.
- Label the bottles or "Pump & Save" bags with the date of expression.

Caution

Do not thaw frozen breastmilk or do not warm breastmilk in a microwave or a pan of boiling water to avoid loss of vitamins, minerals and other important components and to prevent burns.

- In order to preserve the components of the breastmilk, thaw the milk in the refrigerator overnight. Alternatively, you can hold the bottle or "Pump & Save" bag under warm water (max. 37 °C).
- Gently swirl the bottle or "Pump & Save" bag to mix any fat that has separated. Avoid shaking or stirring the milk.

For your child's safety and health

WARNING!

- Always use this product with adult supervision.
- Never use feeding teats as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Keep all components not in use out of the reach of children.
- Do not warm milk in a microwave oven as this may cause uneven heating and could scald your baby.

- Plastic bottles and component parts become brittle when frozen and may break when dropped.
- Also, bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.
- Take appropriate care in handling bottles and components.
- Do not use the breastmilk if bottles or components become damaged.

Material	Bottle: Polypropylene Teat: Silicone
Disposal	According to local regulations

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