Abilar® 10 % Wound Balm

1 tube 20 g

Read these instructions for use carefully before using the product.

The salve is used externally for topical treatment.

What is in these instructions for use

- What Abilar® 10% resin salve is and what it is used for
- Instructions for use of Abilar® resin salve
- Active substance
- Other ingredients
- Storage and stability
- Possible side effects
- Other information

Intended purpose:

The efficacy of Abilar® 10% resin salve is based on the effects of Norway spruce (*Picea abies*) resin. Abilar® 10% resin salve is used for the topical treatment of infected and non-infected wounds, at home and in professional use. The wounds can be acute or chronic; for example, a laceration, abrasion, fissure, burn, pressure ulcer, diabetic ulcer, surgical wound complication or opened abscess.

The raw resin in Abilar® 10% resin salve is purified using a purification method patented by Repolar Pharmaceuticals Oy. The composition, purity and biological and antimicrobial efficacy of the salve is determined from each production batch. The quality system of Repolar Pharmaceuticals Oy is ISO 13485:2016 certified.

Instructions for use of Abilar® 10% resin salve:

Clean the laceration, abrasion or wound and dry it. Apply Abilar® 10% resin salve directly onto the affected area once or twice a day. If necessary, the affected area can be protected with a breathable dressing or plaster. When treating wound cavities or opened abscesses, follow the instructions given by a wound care nurse or doctor. Abilar® 10% resin salve forms a protective layer on the surface of the wound to enhance healing. Avoid applying the salve to the eyes or mouth.

Active substance: Purified Norway spruce (*Picea abies*) resin.

Other ingredients: Petrolatum, Paraffinum liquidum, Alcohol denat., Cera microcristallina, Sorbitan oleate, Cera alba, Hydrogenated castor oil, Stearic acid.

Storage and stability: After opening will remain stable at room temperature for 1 year and unopened for a minimum of 5 years.

12M = Period-after-opening.

Pregnancy and lactation: There is no study data about possible adverse effects during pregnancy and lactation.

Interactions: No clinically significant interactions are known.

Effect on ability to drive: Does not affect the ability to drive and use machines.

Adverse effects: A possible adverse effect is an allergic reaction caused by the resin compounds of the spruce resin occurring as topical irritation of the skin in the treated area. If allergic skin reactions appear, the use of the resin salve must be discontinued. Please report any adverse effects, interactions etc. to the manufacturer.

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